

## Yale University Library Digital Collections

<b>Title</b>	"Brooklyn" Combination Battery
<b>Creator</b>	Brooklyn Dry Cell Battery Company
<b>Collection Title</b>	Medical Instrument Collection
<b>Rights</b>	The use of this image may be subject to the copyright law of the United States (Title 17, United States Code) or to site license or other rights management terms and conditions. The person using the image is liable for any infringement.
<b>Extent of Digitization</b>	Unspecified
<b>Container information</b>	Box 068
<b>Generated</b>	2023-12-21 18:50:15 UTC
<b>Terms of Use</b>	<a href="https://guides.library.yale.edu/about/policies/access">https://guides.library.yale.edu/about/policies/access</a>
<b>View in DL</b>	<a href="https://collections.library.yale.edu/catalog/17316648">https://collections.library.yale.edu/catalog/17316648</a>

---





Instructions for the use of the

**"BROOKLYN" COMBINATION BATTERY.**

To use the galvanic current, connect cords with handles &c. to post marked N & P. Turn on the number of cells desired, from 1 to 6 and regulate the current of each or all cells on the Rheostat. For the Electric Lamps, use 5 cells with the rheostat switch on the second or third button. Whenever said current does not fully illuminate lamp, turn rheostat switch further to the left.

For the Cautey Electrode use 3, 4, 5 or 6 cells in accordance to the heat desired. For cautey work the rheostat switch **MUST** be turned to the fourth last point toward the left.

**FARADIC CURRENT.**

To use the Faradic Current turn upper switch to either post 2 (two cells) or post 3 (three cells), and use current on posts marked 1 to 4 as follows:

Posts 1 & 2 is the primary,	- - - - -	mildest current.
Posts 3 & 4 is the 2nd secondary,	- - - - -	2nd stronger current.
Posts 2 & 3 is the 1st secondary,	- - - - -	3rd stronger current.
Posts 2 & 4 is the entire secondary,	- - - - -	4th stronger current.
Posts 1 & 3 is the primary & 1st secondary,	- - - - -	5th stronger current.
Posts 1 & 4 is the entire coil	- - - - -	6th and strongest current.

For Muscular Contraction use current from posts 1 and 2. For Paralysis and Artificial Respirations use current from posts 1 and 3 or 1 and 4. To relieve pains use current from either posts 3 and 4, 2 and 3 or 2 and 4.

**CAUTION**—Always turn switches to (O) off when battery is not in use. **NEVER** place metal of any kind on the trimmings on base.

**Instructions regards use of Attachments.**

The Universal handles, are for the use of the Electrolytic needle tips and the sponge disk's, both screw on said handles. The Urethral Olives, Rectal Ball Electrode and Uterine cup, all fit on long rubber stem. To renew Lamps, warm the rubber end and withdraw same from holders. In the same manner lamp holders can be sterilized.

**How to use the Masseur with the Faradic Current.**

Connect one conducting cord to end of handle of masseur, and apply to, or have patient hold, either metal or sponge handle. For the face, chest, abdomen and palmer surface of the hand or foot, use the square chamois covered hammer (moist). For Sciatica, Lumbago and for the thigh and neck, use the round chamois covered hammer. For superficial nerves, dorsum of the hand or foot, use the six pointed, metal hammer.

**BROOKLYN DRY CELL BATTERY CO.,**

991 MADISON STREET,

Brooklyn, N. Y.